



人と科学のステキな未来へ

コスモ・バイオ株式会社



BRADLEY PRODUCTS


INSTRUCTIONS FOR THE USE OF THE DAVIDSON[®] SMALL SPECIMEN KIT

Bradley Products, Inc.

1700 West 94th Street — Bloomington, MN 55431-2300

(952)-881-1430, (800)-325-7785, or fax (952)-881-1873

Internet: www.bradleyproducts.com — E-mail: dms@bradleyproducts.com

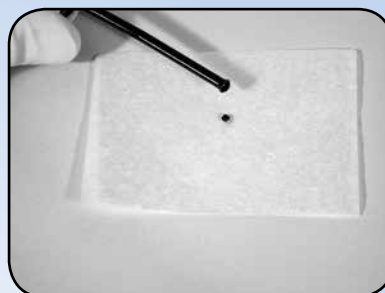




Davidson[®] Small Specimen Kit Instructions


Step 1

Place small specimen biopsy (1-2mm) onto a piece of cassette paper provided.

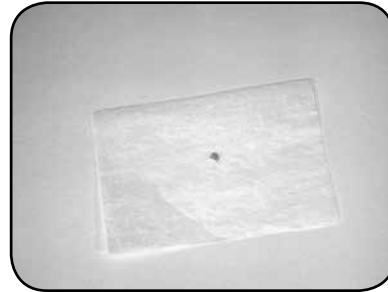




Step 2

Add one drop of Davidson[®] Small Specimen dye to specimen on tissue.

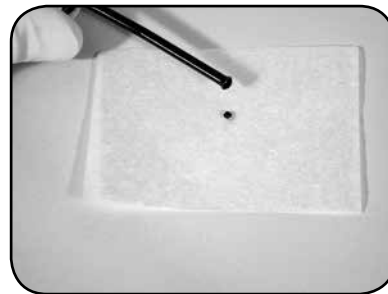

Step 3

Fold cassette paper to secure specimen. Place in cassette. Add sponge spacer. Close cassette.

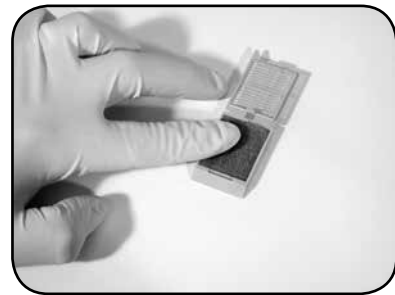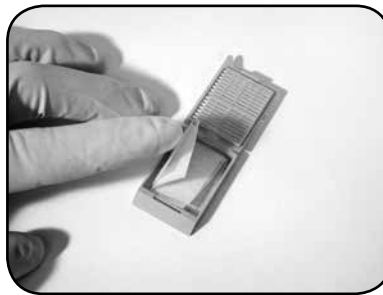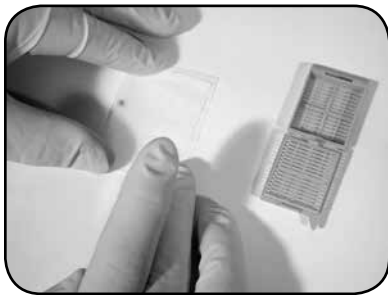



Step 4

Process using standard fixing techniques (alcohol, formalin, etc.)

Step 5

After fixing, unfold cassette paper. Place the colored specimen and paper into an embedding cassette.

Step 6

Embed, section and stain using standard techniques.

NOTE: The Davidson® Small Specimen Kit dyes are primarily designed to keep track of the small specimen through the fixation process. If marking is required (for example, to process two or more small specimens in a single cassette), then either experiment with these dyes or use the more conventional Davidson Marking System® dyes to ensure specimens are adequately marked.



人と科学のステキな未来へ

コスモ・バイオ株式会社



To Order
800-325-7785

Or, you may place your order by:

- **Fax:** 952-881-1873
- **Mail:** Bradley Products, Inc.
1700 West 94th Street
Bloomington, MN 55420
- **Internet:** www.bradleyproducts.com
- **Phone:** 800-325-7785
001-952-881-1430
- **E-mail:** dms@bradleyproducts.com

Payment may be made by:

- **Check** drawn on a U.S. bank
- **Credit Card**
(VISA, MasterCard and Discover)
- **Electronic Funds Transfer**

Terms: Net 30 Days Federal I.D. # 41-0979462

Shipping Charges Are "Prepay and Add"

US orders are "Prepay and Add". UPS Ground, Next Day, and 2nd Day are available. International orders are assessed a \$15.00 handling charge plus the cost of shipping. Most UPS services are available. UPS and FedEx collect are also available. Order by phone, fax, e-mail or via the Internet. Mail-in orders should be sent to the address above. VISA, MasterCard, Discover and American Express are accepted. EFT is accepted for International orders (customer pays en route bank charges). A full color brochure is available at our web site or upon request. Payment Terms are Net 30. US \$\$ only.



Bradley Products, Inc.
1700 W 94th Street, Bloomington, MN 55431
phone: 952-881-1403
toll-free: 800-325-7785
fax: 952-881-1873
e-mail: dms@bradleyproducts.com
internet: www.bradleyproducts.com

Product List (Revised March 2015)

Item Description

Complete Davidson Marking Systems®

Comes with Wood Tray.

For Plastic Tray, add "-P" to item #. (i.e., 2407-P)

2407	7-color System 2 oz. (all 7 colors)
2406	6-color System (green, yellow, black, red, blue, orange)
2401	5-color System (green, yellow, black, red, blue)
2403	3-color System (choose colors)
1207	7-color System 1 oz. (all 7 colors)
1007	7-color System 3cc MINI (3 of ea color)

Individual Dyes

x=color: 1-green, 2-yellow, 3-black, 4-red, 5-blue, 6-orange, 7-violet

1013-(x)	3cc MINI dyes (each color - 40 pieces)
1101-(x)	1 oz. dye (each color)
1163-(x)	2 oz. dye (each color)
3408-(x)	8 oz. dye (each color)

Wood Holding Trays

For Plastic Tray, add "-P" to item #. (i.e., 1877-P)

1877	7-dye Tray - holds 2 oz. bottles
1866	6-dye Tray - holds 2 oz. bottles
1873	5-dye Tray - holds 2 oz. bottles
1833	3-dye Tray - holds 2 oz. bottles
1817	7-dye Tray - holds 1 oz. bottles
1807	21-dye Tray - holds 3cc MINI bottles
1888	5-dye Tray - holds 8 oz. bottles

Applicator Sticks & Brushes

7399	10 1" brushes / 1 16oz wide mouth jar
8553	500 Sticks (ten packs of 50)
8554	Brush Caps for 2 oz. bottles (bag of 10)
8555	Tapered end cotton tipped sticks (250)
8556	Round end cotton tipped sticks (250)
8557	1 inch wide brushes (25)
8558	2 inch wide brushes (25)

Davidson® Cryocup® System / Davidson® Small Specimen Kit

Various	IMPROVED Davidson® Cryocup® System (visit web site or contact us regarding NEW options developed for the Cryocup® System)
6601	Davidson® Small Specimen Kit (Set of green, red & blue 1 oz. bottles with applicator caps, wood tray, and 250 cassette papers.)
6605	Refill packet of 500 cassette papers